

Instructions for the EZ Health™ Oracle™ Lancet Device

The EZ Health™ Oracle™ Lancet Device is used for obtaining capillary blood samples for glucose monitoring or for other tests which require one or two drops of blood. An adjustable tip allows the best depth of skin penetration for each individual user. A drop of blood is most easily obtained by first washing your hands in warm, soapy water.

Directions

1. Unscrew the EZ Health™ Oracle™ Lancet Device tip and insert a new lancet firmly into the carrier. Next pinch the lancet and twist off the protective cover. Replace the threaded tip.
2. The new comfort tip offers 5 different levels of skin penetration.

To select best depth:

- 1-2 for soft or thin skin
- 3 for average skin
- 4-5 for thick or calloused skin

To select a desired depth, move the number to match the arrow

Note:

For even finer adjustment, the arrow may be placed between any two numbers.

3. Next hold the tip in one hand and pull on the sliding barrel with the other hand. The device will be cocked when a click is felt as the ends are pulled apart.
4. Release the sleeve. It will automatically move back to its original position near the trigger hub.
5. Set aside the EZ Health™ Oracle™ Lancet Device and wait a few seconds for a blood drop to form.
6. The flow of blood will be helped by keeping the hand warm, by lowering the hand to waist level, and by gentle massage of the finger.
7. For safety reasons, and to prevent cross-contamination, discard the used lancet into an appropriate sharps or biohazard container. Never allow another person to use a lancet that has already been used. If the lancing device is to be used by another person, the unit must be provided with a new tip and sterile lancet.
8. Lancet removal: Unscrew the EZ Health™ Oracle™ Lancet Device tip. Carefully stick the protective disk cover back on the used lancet. Grip the holder firmly and pull the lancet safely out of the holder. Screw the tip back on the EZ Health™ Oracle™ Lancet Device.

Taking Care of your EZ Health™ Oracle™ Lancet Device:

Use alcohol or soap and water to wipe the outside of the lancing device. For more complete cleaning, remove the tip, wash it in warm water and rinse well. It can be disinfected with 70% rubbing alcohol. (Hospitals should consult their own infection control protocol.)

Limited Warranty

Tremblay Harrison, Inc. warrants that the EZ Health™ Oracle™ Lancet Device will function effectively for a period of five (5) years from the date of purchase. A malfunctioning device should be returned to your supplier or to Tremblay Harrison, Inc. for replacement.

Manufacturer: Tremblay Harrison Inc. 1 (866) 829-7926
1684 Dufferin Street
Toronto, ON M6H 3M1

